Suomi

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English

Qufora® IrriSedo Irrigation System Using the Qufora® IrriSedo Irrigation System makes it possible to perform rectal irrigation. Please read both pages of this instruction before using the Qufora® IrriSedo Irrigation System.

Application
The Qufora® IrriSedo Irrigation System is used to prevent chronic fecal incontinence and/or constipation or for those who spend a long time on bowel management. It should be used regularly - usually daily or every other day to provide the best effect.

Before starting rectal irrigation
Use only after assessment, approval and full instruction by a healthcare professional who is specialised in the procedure. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Indications for using rectal irrigation
Indications that include the following conditions and symptoms may benefit:

• Neurogenic bowel dysfunction, e.g. spinal disease or ulcerative colitis).
• Active perianal sepsis (fistula or abscess).
• Rectal or colonic surgery within the past 6 months.
• Diarrhoea of unknown aetiology.
• Fecal impaction/ rectal constipation.
• Severe autonomic dysreflexia.
• Severe diverticulosis or diverticular abscess.
• Abdominal or pelvic radiotherapy.
• Known diverticular disease.
• Long term steroid therapy.
• Anti-coagulant therapy.
• Low blood sodium.
• Pelvic malignancy.
• Colonic biopsy within the past 3 months.
• Use of rectal medications for other diseases which may be diluted by irrigation.
• Congestive cardiac failure.

Protect the product from direct sunlight and do not expose to heavy pressure.

The product should be kept out of reach of children.

Further information about this product can be found on www.mbh-international.com.

Before starting rectal irrigation
Use only after assessment, approval and full instruction by a healthcare professional who is specialised in the procedure. A digital rectal examination is mandatory prior to your first irrigation. You should be supervised by an experienced healthcare professional during your first irrigation.

Indications for using rectal irrigation
Indications that include the following conditions and symptoms may benefit:

• Neurogenic bowel dysfunction, e.g. spinal disease or ulcerative colitis).
• Active perianal sepsis (fistula or abscess).
• Rectal or colonic surgery within the past 6 months.
• Diarrhoea of unknown aetiology.
• Fecal impaction/ rectal constipation.
• Severe autonomic dysreflexia.
• Severe diverticulosis or diverticular abscess.
• Abdominal or pelvic radiotherapy.
• Known diverticular disease.
• Long term steroid therapy.
• Anti-coagulant therapy.
• Low blood sodium.
• Pelvic malignancy.
• Colonic biopsy within the past 3 months.
• Use of rectal medications for other diseases which may be diluted by irrigation.
• Congestive cardiac failure.

Protect the product from direct sunlight and do not expose to heavy pressure.

The product should be kept out of reach of children.

Further information about this product can be found on www.mbh-international.com.

Irrigation should not be used under the following circumstances (absolute contraindications):

• Known anal or colorectal stenosis.
• Colorectal cancer pre surgical removal.
• Acute inflammatory bowel disease.
• Acute diverticulitis.
• Within 3 months of anal or colorectal surgery.
• Within 4 weeks of endoscopic polypectomy.
• Ischaemic colitis.

Since the list is not exhaustive a healthcare professional should always consider individual patient factors as well.

Use rectal irrigation only after careful assessment by a qualified medical practitioner under the following circumstances (relative contraindications):

• Pregnancy or planned pregnancy or breast feeding.
• Children under 3 years of age, consid-

ered the recommended usage time, pump should be replaced.

The cone is a single use item.
Liability
MBH-International A/S accepts no liability for any injury or other loss that may arise if this product is used in any way contrary to the recommendations of MBH-International A/S.

Handling and storage
Qufora® IrriSedo Irrigation System should be stored at temperatures between 0 and 25 °C.

Protect the product from direct sunlight and do not expose to heavy pressure.

The product should be kept out of reach of children.

Further information about this product can be found on www.mbh-international.com.

GB Qufora® IrriSedo Mini System **Instruktion for Use**
SE Qufora® IrriSedo Minisystem **Bruksanvisning**
DE Qufora® IrriSedo Minisystem **Gebrauchsanweisung**
DK Qufora® IrriSedo Minisystem **Brugsanvisning**
FI Qufora® IrriSedo Minisysteem **Brugsanwijzing**
NL Qufora® Mini-järjestelmä **Käyttöohje**
NO Qufora® IrriSedo Minisystem **Bruksanvisning**
ES Sistema Qufora® IrriSedo **Minisistema Instrucciones de uso**
FR Mini-système Qufora® IrriSedo **Mode d'emploi**
IT Sistema Qufora® IrriSedo **Minisistema Istruzioni d'uso**
IL Qufora® IrriSedo **מימון למערכת מיני**
PT-BR Sistema de Mini Qufora® IrriSedo **Instruções para uso**

Qufora® is a registered trademark of MBH International A/S. All rights reserved MBH-International A/S, 3450 Ålensd, Denmark.

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