

Sage Oral Swab with Corinz™ Oral Rinse

REF 6583-X



Packaging information

Contents:

- (2) Applicator Swabs with sodium bicarbonate, (1) 7 mL Corinz Oral Rinse.

Indications of use: Aids in the removal of debris and oral secretions.

Instructions: Burst solution packet with thumbs. Peel package open. Clean mouth for 1 minute. Instruct to expectorate. Discard according to hospital protocol and local regulations. Repeat as needed/directed.

Caution: Do not allow patient to bite down on the oral care tool. Use a bite block if patient has altered levels of consciousness or cannot comprehend commands. Use caution with children and unresponsive individuals. Failure to follow these safety precautions may damage the device and present a choking/aspiration hazard. May not function as intended/potential risk of cross-contamination if device is reused.

Not made with natural rubber latex.

Applicator Swab: MDD 93/42/EEC **CE**

Corinz Oral Rinse: EC Cosmetic Regulation 1223/2009

Specifications

Product name	Sage Oral Swab with Corinz Oral Rinse
Product code	6583-X
Description	Aids in the removal of debris and oral secretions
Manufacturer accreditation	ISO 13485:2016
Expiration date	24 months from date of manufacture
Length of Applicator Swab	15.9 cm
Amount of solution in Corinz Oral Rinse	7 mL
Case quantity	100

Features and benefits

Applicator Swab

Applicator Swab	<ul style="list-style-type: none"> • Helps remove debris and secretions between brushings
Treated with sodium bicarbonate	<ul style="list-style-type: none"> • Sodium bicarbonate allows for enhanced cleaning and refreshes the oral cavity
Soft foam heads	<ul style="list-style-type: none"> • Gentle to fragile tissue

Sage Oral Swab with Corinz™ Oral Rinse

Corinz Oral Rinse ingredients

Water (aqua)
Glycerin
Xylitol
Flavor (aroma)
Hydroxyethylcellulose
Potassium sorbate
Cetylpyridinium chloride
Sodium saccharin
Citric acid
Polysorbate 20
Polysorbate 80
Menthol

Sodium bicarbonate ingredients

Water (aqua)
Sodium bicarbonate
Cellulose gum
Sodium lauryl sulfate
Flavor (aroma)
Sodium saccharin
Sodium benzoate



Sage Products LLC
3909 Three Oaks Road
Cary, IL 60013 USA
t: 0808 189 0456

stryker.com

SMACC # 23796
26985-UK
Copyright © 2020 Stryker
02/2020

This document is intended solely for the use of healthcare professionals.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, Corinz. All other trademarks are trademarks of their respective owners or holders.

The products depicted are CE marked in accordance with applicable EU Regulations and Directives.

This document is not intended for distribution outside EU and EFTA.