

Sage Oral Care System q4°



REF 6964-X

Packaging information

Contents:

- (2) packages of (1) Untreated Suction Toothbrush, 7 mL Sage Oral Solution, (1) Applicator Swab.
- (4) packages of (1) Suction Swab with sodium bicarbonate, 7 mL Corinz™ Oral Rinse

Indications of use:

Suction Toothbrush: Aids in the removal of dental plaque, debris, and oral secretions.

Suction Swab: Aids in the removal of oral secretions and debris.

Instructions:

Suction Toothbrush: Burst solution packet with thumbs. Peel package open. Attach Suction Toothbrush to suction line. Brush teeth and oral cavity for 1 minute. To suction, place thumb over port. Rinse to clear tubing after use. Discard. Use Applicator Swab if additional cleansing is needed.

Sage Oral Solution: Do not rinse with water or other mouthwashes, brush teeth or eat immediately after using Sage Oral Solution. Do not swallow. To minimize discoloration of teeth, brush and floss daily. STORE ABOVE FREEZING (0°C or 32°F).

Suction Swab: Burst solution packet with thumbs. Peel package open. Attach Suction Swab to suction line. Clean mouth for 1 minute. To suction, place thumb over port. Rinse to clear tubing after use. Discard.

Corinz Oral Rinse: Children under 3 years of age: Consult a dentist or doctor. Children under 12 years of age: Supervise use. Use under the guidance of a healthcare professional. Keep out of reach of children.

Caution:

Do not allow patient to bite down on the oral care tool. Use a bite block if patient has altered levels of consciousness or cannot comprehend commands. Use caution with children and unresponsive individuals. Failure to follow these safety precautions may damage the device and present a choking/aspiration hazard. May not function as intended/potential risk of cross-contamination if device is reused.

Not made with natural rubber latex.

Specifications

Product name	Sage Oral Care System q4°
Product code	6964-BP
Description	Daily kit to provide oral hygiene including tools for each 4-hour nursing shift
Manufacturer accreditation	ISO 13485:2016
Expiration date	24 months from date of manufacture
Length of Suction Toothbrush	18.4 cm
Length of each bristle	8 mm
Length of Suction Swab	18.4 cm
Length of Applicator Swab	15.9 cm
Amount of solution in each burst packet	7 mL
Case quantity	20

Sage Oral Care System q4°

Features and benefits

Suction Toothbrush

Ultra-soft nylon bristles	• Gently removes dental plaque, debris, and oral secretions known to harbor respiratory pathogens
Over 3.100 bristles or "contact points"	• Cleans and removes plaque
Three suction ports	• Avoids clogging and keeps open path for debris
Connects directly to standard suction lines	• Helps provide maximum suction strength
Angled handle	• Provides cleansing of the back of the oral cavity

Suction Swab

Suction Swab	• Helps remove debris and secretions between brushings
Treated with sodium bicarbonate	• Sodium bicarbonate allows for enhanced cleaning and refreshes the oral cavity
Soft foam heads	• Gentle on fragile tissue

Sage Oral Solution ingredients

Water (aqua)
 Alcohol
 Glycerin
 PEG-40 sorbitan diisostearate
 Flavor (aroma)
 Chlorhexidine digluconate
 Sodium saccharin
 Blue 1 (CI 42090)

Sodium bicarbonate ingredients

Water (aqua)
 Sodium bicarbonate
 Cellulose gum
 Sodium lauryl sulfate
 Flavor (aroma)
 Sodium saccharin
 Sodium benzoate

Corinz™ Oral Rinse ingredients

Water (aqua)
 Glycerin
 Xylitol
 Flavor (aroma)
 Hydroxyethylcellulose
 Potassium sorbate
 Cetylpyridinium chloride
 Sodium saccharin
 Citric acid
 Polysorbate 20
 Polysorbate 80
 Menthol

This document is intended solely for the use of healthcare professionals.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders.

The products depicted are CE marked in accordance with applicable EU Regulations and Directives.

This document is not intended for distribution outside EU and EFTA.

Sage Products LLC
 3909 Three Oaks Road
 Cary, IL 60013 USA
 t: 0808 189 0456

sage-products.co.uk

26297-UK
 Copyright © 2018 Stryker