


# Sage Untreated Oral Swab

**REF 5602UT-X**

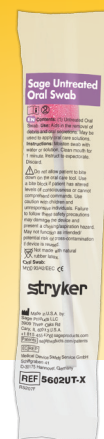

## Packaging information

### Contents:

- (1) Untreated Oral Swab

**Use:** Aids in the removal of debris and oral secretions. May be used to apply oral care solutions.

**Instructions:** Moisten swab with water or solution. Clean mouth for 1 minute. Instruct to expectorate. Discard according to hospital protocol and local regulations.

### Caution:

Do not allow patient to bite down on the oral care tool. Use a bite block if patient has altered levels of consciousness or cannot comprehend commands. Use caution with children or unresponsive individuals. Failure to follow these safety precautions may damage the device and present a choking/aspiration hazard. May not function as intended/potential risk of cross-contamination if device is reused.

For single use only.

Not made with natural rubber latex.

## Specifications

Product name	Sage Untreated Oral Swab
Product code	5602UT-X
Description	Applicator swab designed to aid in the removal of debris and oral secretions and apply oral care solutions
Manufacturer accreditation	ISO 13485:2016
Length of Swab	11 cm
Case quantity	1000

## Features and benefits

### Swab

Swab	<ul style="list-style-type: none"> <li>• Helps remove debris and oral secretions between brushings</li> </ul>
Soft foam heads	<ul style="list-style-type: none"> <li>• Gentle on delicate oral tissues</li> </ul>

### This document is intended solely for the use of healthcare professionals.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, Toothette Oral Care, Toothette. All other trademarks are trademarks of their respective owners or holders.

The products depicted comply with EC directives and regulations applicable within the European Union.

This document is not intended for distribution outside EU and EFTA.



Sage Products LLC  
3909 Three Oaks Road  
Cary, IL 60013 USA  
800 323 2220 ; +1 815 455 4700

**stryker.com**

SMACC # 25537  
27364-UK  
Copyright © 2020 Stryker  
06/2020