

Disposable Bite Block for Oral Control



REF 4000-X

Packaging information

Contents:

- (1) Disposable Bite Block

Instructions: Use a bite block when performing oral care on patients with altered levels of consciousness. Place padded part of bite block in mouth between the teeth. If teeth have clamped shut, do not attempt to force mouth open or force bite block in. Discard according to hospital protocol and local regulations.

Caution:

May not function as intended/potential risk of cross-contamination if device is reused. Ensure foam is intact after use. If not, remove any particles from oral cavity.

Not made with natural rubber latex.

For single use only.

Specifications

Product name	Disposable Bite Block
Product code	4000-X
Description	Padded bite block used on patients with altered levels of consciousness
Manufacturer accreditation	ISO 13485:2016
Length of Bite Block	15.9 cm
Width of Bite Block	3.2 cm
Composition of Foam Head	Polyethylene foam
Case quantity	144

This document is intended solely for the use of healthcare professionals.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker. All other trademarks are trademarks of their respective owners or holders.

The products depicted are CE marked in accordance with applicable EU Regulations and Directives.

This document is not intended for distribution outside EU and EFTA.



Sage Products LLC
3909 Three Oaks Road
Cary, IL 60013 USA
800 323 2220 ; +1 815 455 4700

stryker.com

SMACC # 25538
27365-UK
Copyright © 2020 Stryker
06/2020