

Covered Yankauer Suction Handle and Y-Connector Non-sterile

REF 6639-X



Packaging information

Contents:

- (1) Covered Yankauer, (1) Suction Handle, (1) Y-Connector

Use: Aids in the removal of oral secretions.

Instructions: Attach tubing directly to suction port. If additional suction port is needed, attach Y-Connector to suction port. Attach tubing to Y-Connector. **Important:** Both ports on the Y-Connector must be utilized or a loss of suction power can occur. Attach Suction Handle with Yankauer to tubing. Retract sleeve. Slide cap down until it locks in place. To suction, slide switch ON. **Important:** when not in use, make sure the ON/OFF switch is in the OFF position to prevent loss of suction power. Pull sleeve over Yankauer tip to cover. After use, rinse with sterile saline or appropriate solution until tubing is clear. Remove Yankauer and Y-Connector (if used) and discard after 24 hours, or according to hospital protocol and local regulations.

Caution:

Single patient use only by a healthcare professional. Do not allow patient to bite down on the oral care tool. Use a bite block if patient has altered levels of consciousness or cannot comprehend commands. Use caution with children and unresponsive individuals. Failure to follow these safety precautions may damage the device and present a choking /aspiration hazard. May not function as intended/potential risk of cross-contamination if device is reused.

Not made with natural rubber latex.

Specifications

Product name	Covered Yankauer, Suction Handle and Y-Connector
Product code	6639-X
Description	Covered Yankauer with Suction Handle and Y-Connector
Manufacturer accreditation	ISO 13485:2016
Length of Yankauer	26.3 cm
Case quantity	60

This document is intended solely for the use of healthcare professionals.

A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to use a particular product when treating a particular patient. Stryker does not dispense medical advice and recommends that healthcare professionals be trained in the use of any particular product before using it.

The information presented is intended to demonstrate the breadth of Stryker product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any Stryker product.

Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker products in your area.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: Stryker, Q-Care. All other trademarks are trademarks of their respective owners or holders.

The products depicted comply with EC directives and regulations applicable within the European Union. Sage suction medical devices including Suction Toothbrushes, Suction Swabs, Yankauer, suction handle, and Y-Connector are class IIA medical devices (CE 2797).

This document is not intended for distribution outside EU and EFTA.



Sage Products LLC
3909 Three Oaks Road
Cary, IL 60013 USA
800 323 2220 ; +1 815 455 4700

stryker.com

SMACC # 25539
27366-UK
Copyright © 2020 Stryker
06/2020