



## Instructions for use


Oasis<sup>®</sup> Bed Bathing Washcloths are impregnated with a unique combination of skin cleansers to help remove sweat, dirt, urine and faeces from the skin. The large, soft washcloths are intended to be used individually, with one single-use cloth per anatomical region.

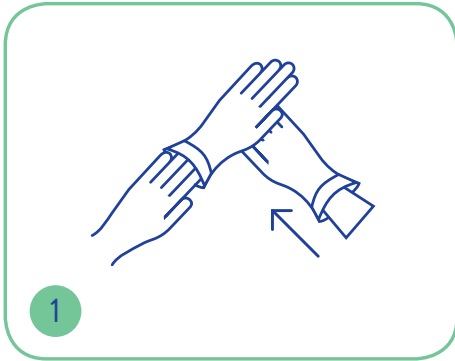


Apply protective gloves

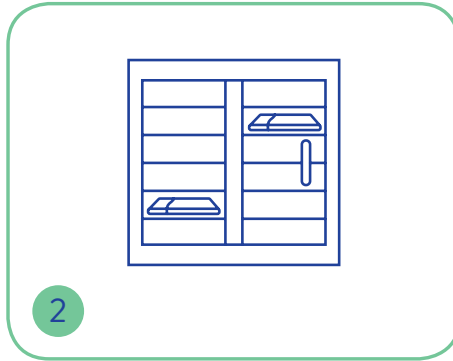


Remove wipes/washmitts from the dedicated Oasis Warmer

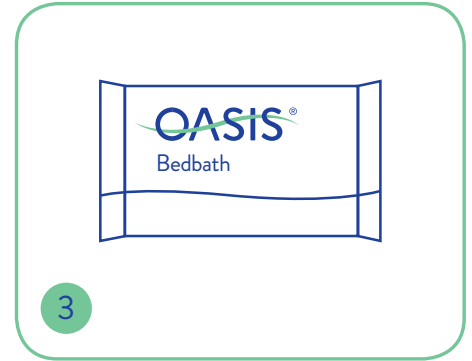


Or use at room temperature

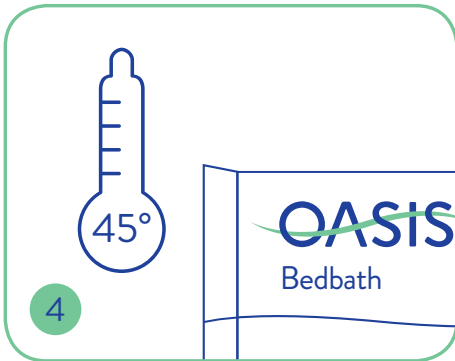


Note how many wipes/washmitts are in each pack and check temperature

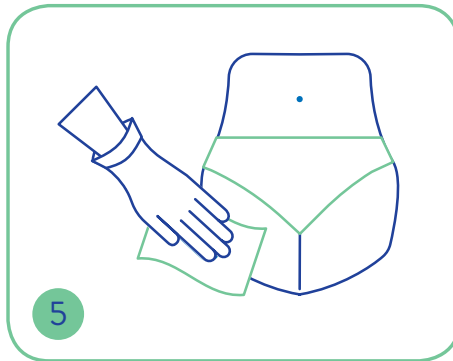

Use one wipe/washmitt to per anatomical region

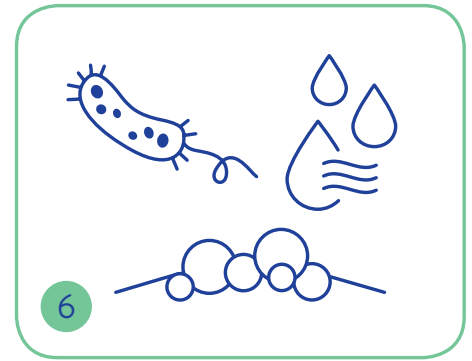


The wipes/washmitts will cleanse and moisturise the skin

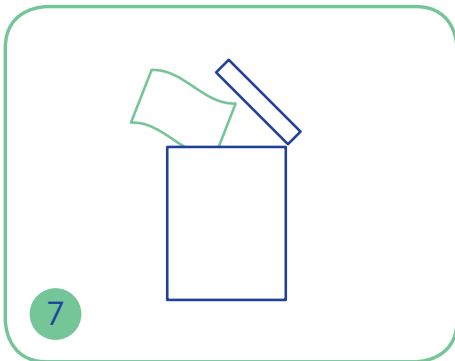

Dispose of used wipes/washmitt

Description	Size	Pack Qty	Case Qty	Code
Oasis <sup>®</sup> Bed Bath Scented	33 x 22cm	10	24	RSC901
Oasis <sup>®</sup> Bed Bath Unscented	33 x 22cm	10	24	RSC902N
Oasis <sup>®</sup> Bed Bath Scented	33 x 22cm	8	30	RSC912
Oasis <sup>®</sup> Bed Bath Unscented	33 x 22cm	8	30	RSC913N
Oasis <sup>®</sup> Bed Bath Unscented	33 x 22cm	5	48	RSC910N
Oasis <sup>®</sup> Washmitt Scented	-	8	30	RSC604A
Oasis <sup>®</sup> Washmitt Unscented	-	8	30	RSC605A

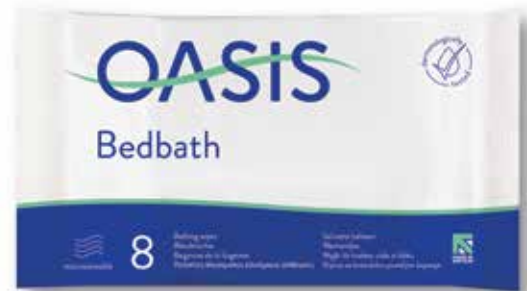


Perfumed and unperfumed options available

# OASIS<sup>®</sup>


## Shampoo Cap

## Instructions for use

Oasis<sup>®</sup> Shampoo Caps are pre-impregnated with a mild shampoo and conditioner. The massaging action releases shampoo and conditioner into the hair and hydrophilic (water-loving) fabric absorbs oil and dirt retaining it within the cap.

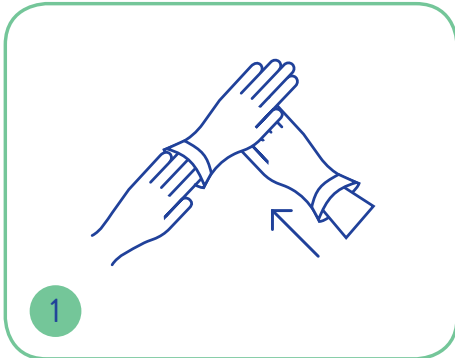


1 Apply protective gloves

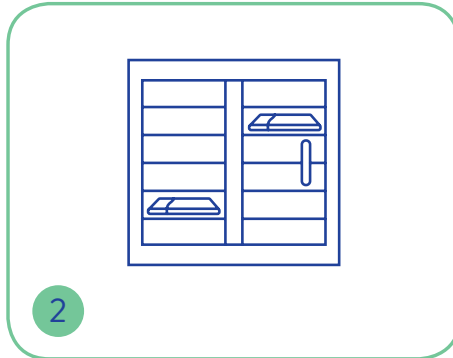


2 Remove shampoo cap from the dedicated Oasis Warmer




3 Or use at room temperature

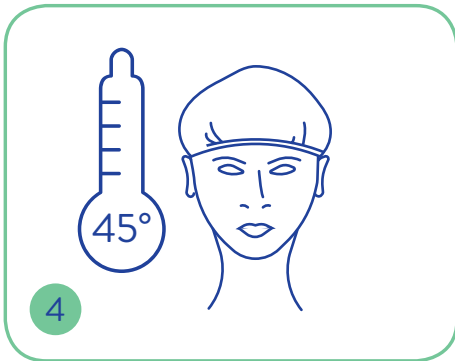

4 Check temperature prior to placing on the user's head, covering all of their hair

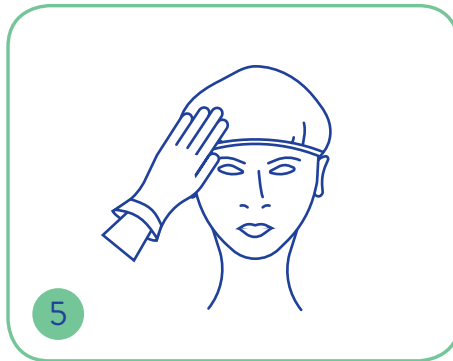


5 Massage thoroughly until hair is saturated.

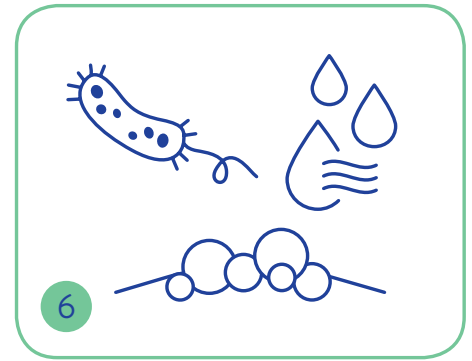

6 The shampoo cap will cleanse and condition the hair



7 Remove the shampoo cap, and dry hair using a towel if excessively wet

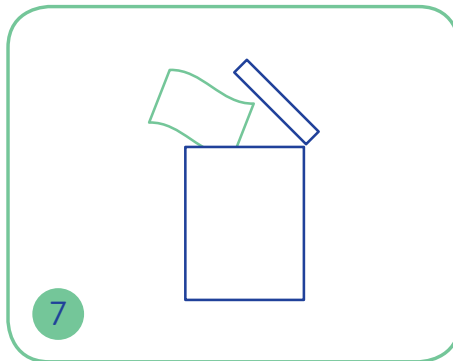

7 Dispose of used shampoo cap

Description	Pack Qty	Case Qty	Code
Oasis <sup>®</sup> Shampoo Cap	1	30	RSC905
Oasis <sup>®</sup> Shampoo Cap Unscented	1	30	RSC905UP
Oasis <sup>®</sup> Shampoo Cap (Economy)	1	30	RSC965
Oasis <sup>®</sup> Shampoo Cap (Economy) Unscented	1	30	RSC965UP



Perfumed and unperfumed options available